




Canada's Leading Independent Optical Lab
PLASTIC PLUS
NO ONE DOES DIGITAL BETTER

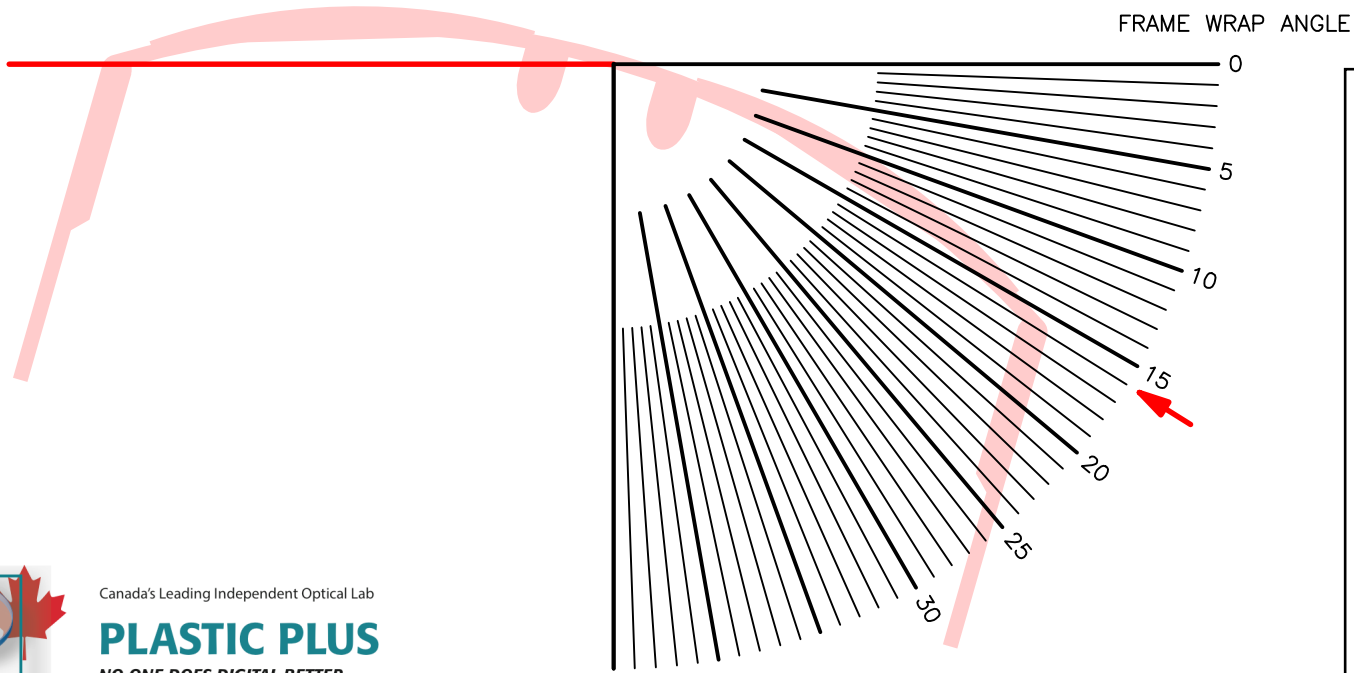

HOW TO MEASURE FRAME WRAP / ZTILT
Place the frame, top side down, over the hi-lited red frame image. Ensure the center of the frame's front is directly over the "T". Record the angle of frame wrap from the extreme right side of where the lens ends, not where the frame ends.

PLEASE NOTE:
This chart will not work as a measurement tool unless printed at exactly 1 to 1 scale. Be sure that your print options are set to Page Scaling: None
DO NOT select "Fit to Printable Area"

1 inch



Office

